

英文

EU UM JUN08

PEN USER MANUAL

FORTEO

FORTEO, 20 microgram (μg) 80 microlitres solution for injection, in pre-filled pen



Instructions for Use

Before you use your new FORTEO, please read the section Instructions for Use completely. Follow the directions carefully when using the FORTEO.

Also read the package leaflet provided.

Your FORTEO contains 28 days of medication.

FORTEO Parts*

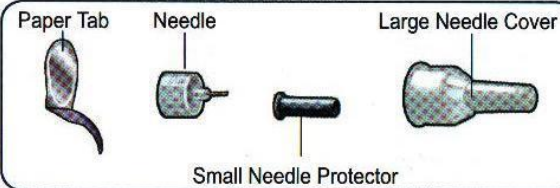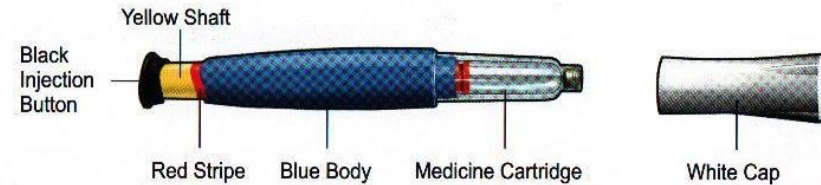


*Needles not included. Becton, Dickinson and Company pen needles 29 to 31 gauge (diameter 0.25-0.33mm) and 12.7, 8 or 5mm length can be used. Ask your healthcare professional which needle gauge and length are best for you.

Always wash your hands before every injection. Prepare the injection site as directed by your healthcare professional.

1

Pull off white cap

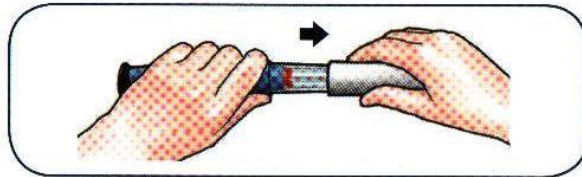


2


Attach new needle




Pull off paper tab.

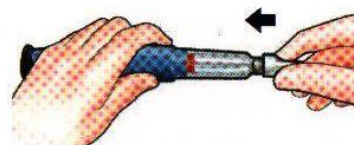

Push needle straight onto medicine cartridge.

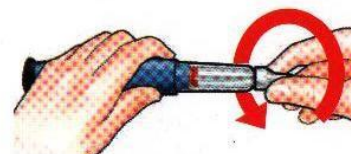

Screw on needle until firmly attached.

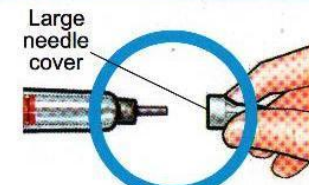


Pull off large needle cover and save it.

3


Set dose

Pull out black injection button until it stops.

If you cannot pull out the black injection button see Troubleshooting, Problem E.

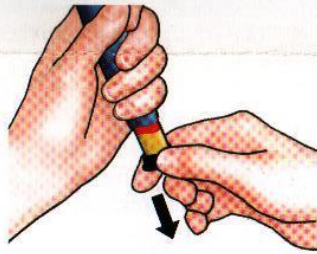

Check to make sure red stripe shows.

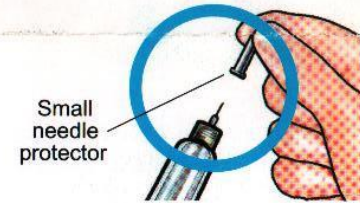

Pull off small needle protector and throw away.

4

Inject dose




Gently hold a fold of skin from thigh or abdomen and insert needle straight into skin.



Push in black injection button until it stops. Hold it in and count to 5 s-l-o-w-l-y. Then pull the needle from skin.

5

Confirm dose

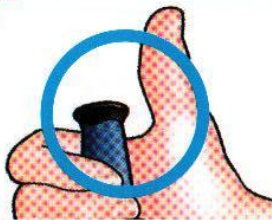


After completing the injection: Once the needle is removed from the skin, check to make sure the black injection button is all the way in. If the yellow shaft does not show, you have completed the injection steps correctly.




You should NOT see any of the yellow shaft. If you do and have already injected, do not inject yourself a second time on the same day. Instead, you MUST reset FORTEO (see Troubleshooting Problem A).

6

Remove needle

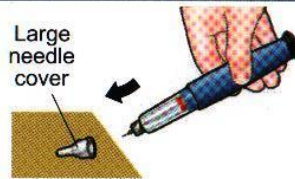


Put large needle cover on needle.

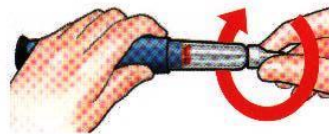

Unscrew the needle all the way by giving the large needle cover 3 to 5 complete turns.



Pull off needle and throw away as directed by your healthcare professional.

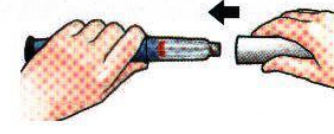

Push white cap back on. Place FORTEO in the refrigerator immediately after use.

IMPORTANT

Lilly



FORTEO

FORTEO, 20 microgram (μg) 80 microlitres
solution for injection, in pre-filled pen

Troubleshooting

Problem

Solution

- A.** The yellow shaft is still showing after I push in the black injection button. How do I reset my FORTEO?



- To reset the FORTEO, follow the steps below.
- 1) If you have already injected, **DO NOT** inject yourself a second time on the same day.
 - 2) Remove the needle.
 - 3) Attach a new needle, pull off the large needle cover and save it.
 - 4) Pull out the black injection button until it stops. Check to make sure the red stripe shows.
 - 5) Pull off the small needle protector and throw away.
 - 6) Point the needle down into an empty container. Push in the black injection button until it stops. Hold it in and count to 5 s-l-o-w-l-y. You may see a small stream or drop of fluid. When you have finished, the black injection button should be all the way in.
 - 7) If you still see the yellow shaft showing, please contact your healthcare professional.
 - 8) Put the large needle cover on needle. Unscrew the needle all the way by giving the needle cover 3 to 5 complete turns. Pull off needle cover and throw away as directed by your healthcare professional. Push the white cap back on, and place FORTEO in the refrigerator.

You can prevent this problem by always using a NEW needle for each injection, and by pushing the black injection button all the way in and counting to 5 s-l-o-w-l-y.

- B.** How can I tell if my FORTEO works?

→ The FORTEO is designed to inject the full dose every time it is used according to the directions in the section Instructions for Use. The black injection button will be all the way in to show that the full dose has been injected from the FORTEO.

Remember to use a new needle every time you inject to be sure your FORTEO will work properly.

- C.** I see an air bubble in my FORTEO.

→ A small air bubble will not affect your dose nor will it harm you. You can continue to take your dose as usual.

- D.** I cannot get the needle off.

-
- 1) Put the large needle cover on the needle.
 - 2) Use the large needle cover to unscrew the needle.
 - 3) Unscrew the needle all the way by giving the large needle cover 3 to 5 complete turns.
 - 4) If you still cannot get the needle off, ask someone to help you.

E. What should I do if I cannot pull out the black injection button?



Change to a new FORTEO to take your dose as directed by your healthcare professional. This indicates that you have now used all the medication that can be injected accurately even though you may still see some medication left in the medicine cartridge.

Cleaning and Storage

Cleaning Your FORTEO

- Wipe the outside of the FORTEO with a damp cloth.
- Do not place the FORTEO in water, or wash or clean it with any liquid.

Storing Your FORTEO

- Refrigerate the FORTEO immediately after every use. Read and follow the instructions in the Information for the Patient leaflet on how to store medication.
- Do not store the FORTEO with a needle attached as this may cause air bubbles to form in the medicine cartridge.
- Store the FORTEO with the white cap on.
- Never store the FORTEO in the freezer.
- If the medication has been frozen, throw the device away and use a new FORTEO.
- If the FORTEO has been left out of the refrigerator, do not throw the pen away. Place the pen back in the refrigerator and contact your healthcare professional.

Disposal of Pen Needles and Device

Disposal of Pen Needles and FORTEO Device

- Before disposing of the FORTEO device, be sure to remove the pen needle.
- Dispose of your FORTEO and used needles as directed by your healthcare professional.
- Dispose of the device 28 days after first use.

Other Important Notes

- The FORTEO contains 28 days of medication.
- Do not transfer the medication into a syringe.
- Write down your first injection date on a calendar.
- Read and follow the instructions in the Information for the Patient leaflet for using medication.
- Check the FORTEO label to make sure you have the correct medication and that it has not expired.
- Contact your healthcare professional if you notice any of the following:
 - The FORTEO appears damaged.
 - The medication is NOT clear, colourless and free of particles.
- Use a new needle for each injection.
- During injection, you may hear one or more clicks – this is a normal device operation.
- Do not share your medication.
- The FORTEO is not recommended for use by the blind or visually impaired persons without the assistance of a person trained in the proper use of the device.
- Keep the FORTEO out of the reach and sight of children.

If you have questions or need help with your FORTEO delivery device, contact "Patient Education Call Center" at 0800889900 or your healthcare provider.

**Manufactured by Lilly France, F-67640 Fegersheim, France
for Eli Lilly and Company.**

RP-09-117